

Step 1: Prepare a reference sample using the original paper.



Step 2: Insert SONOMED paper into printer to do a test print.

Step 3: Adjust brightness levels between 12 o'clock & 2 o'clock positions. Print test copy & compare to original until test copy matches the original. To fine tune, adjust the contrast levels.

Step 1 : Prepare a reference sample using the original paper.



Step 2 : Insert Sonomed paper into the printer to do a test print.



Step 3 : Adjust brightness levels between 12 o'clock & 2 o'clock positions. Print test copy & compare to original until test copy matches the original. To fine tune, adjust the contrast levels.

